




SAMPLES. PATHOGENS. DECISIONS.

CULTURE OF CONFIDENCE

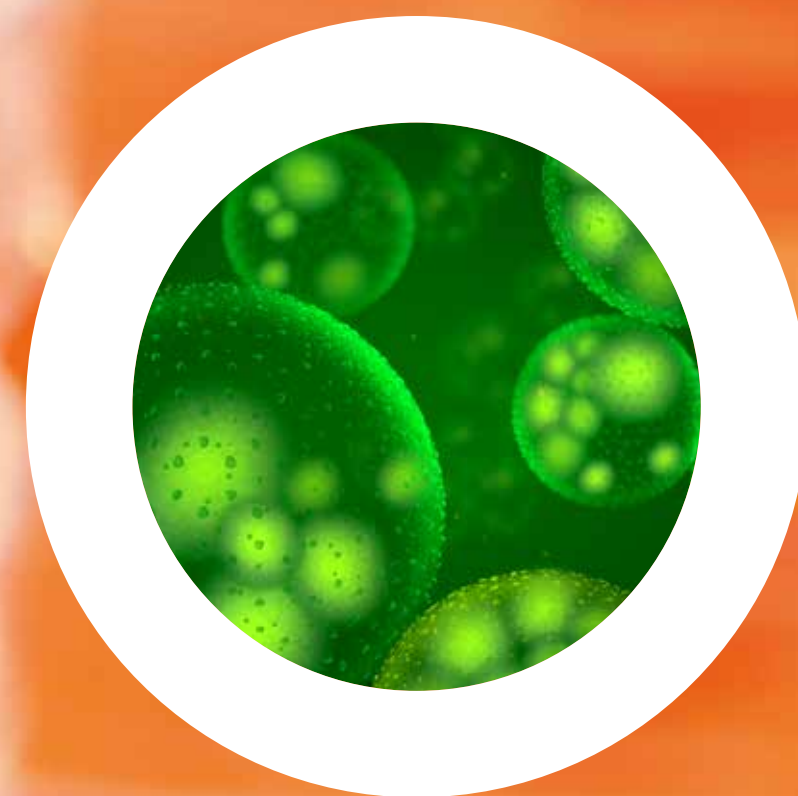

PIONEERING DIAGNOSTICS

The background of the slide is a grayscale image of several petri dishes. One dish in the center is highlighted with a vibrant orange overlay, which also serves as the background for the main title text.

LEADING THE CHARGE


IN CULTURE IDENTIFICATION

CULTURE
OF
CONFIDENCE

CULTURE
OF
QUALITY

CULTURE
OF
SERVICE

CULTURE
OF
INNOVATION

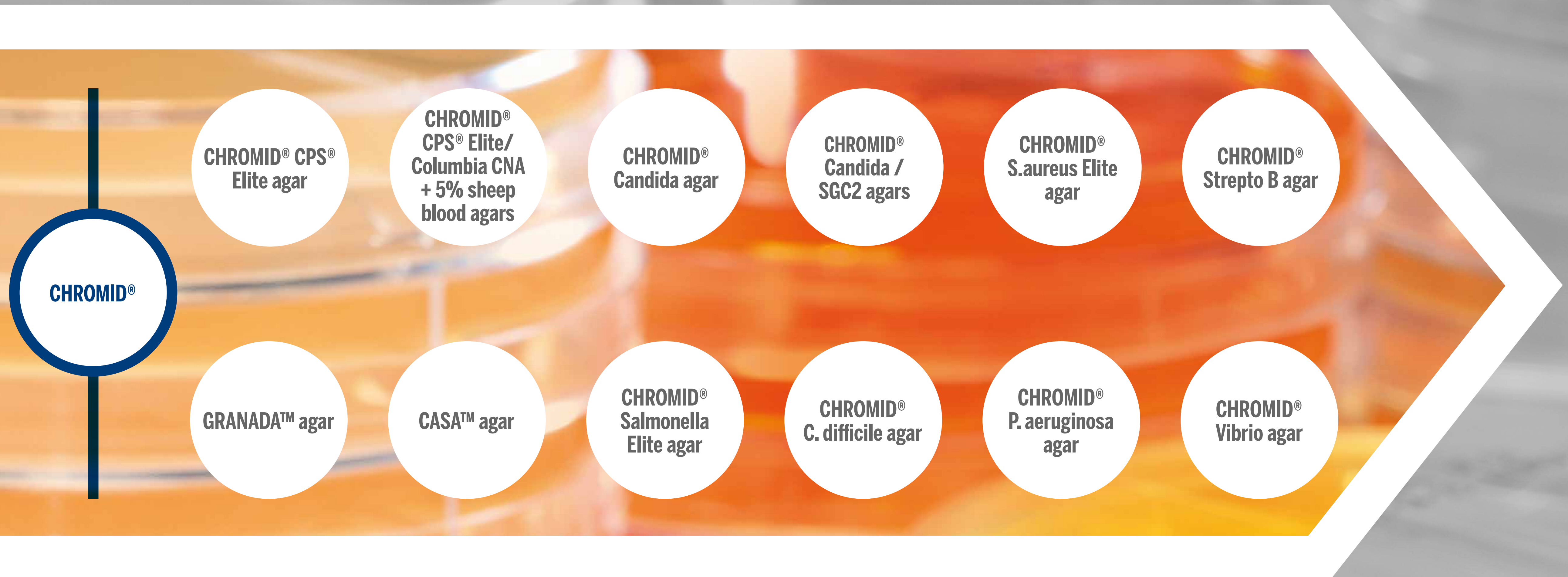


SAMPLE
TYPES

BIBLIO-
GRAPHY

Contact your local representative for local availability

CULTURE OF CONFIDENCE

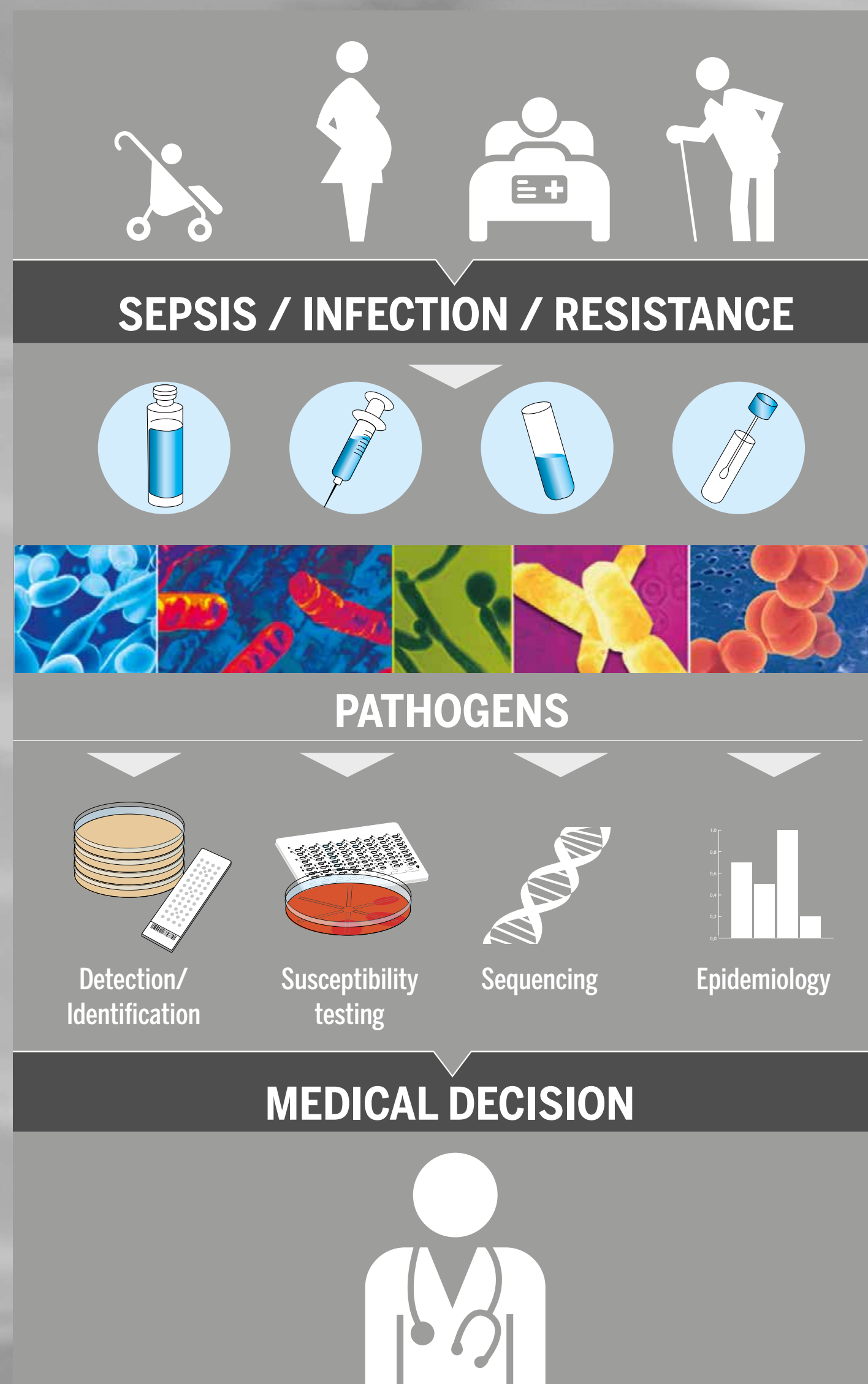

Count on us for a complete diagnostic solution for better patient outcomes and reduced costs

World leader in microbiology

- Over 50 years experience
- Solutions recognized by the international scientific community
- Products tested and approved worldwide

bioMérieux: A partner that cares

- A full range of services behind each medium
- Constant innovation: over 20 different chromogenic media
- 5 media only available from bioMérieux

CULTURE
OF
CONFIDENCE

CULTURE
OF
QUALITY

CULTURE
OF
SERVICE


CULTURE
OF
INNOVATION



SAMPLE
TYPES

BIBLIO-
GRAPHY

CULTURE OF CONFIDENCE

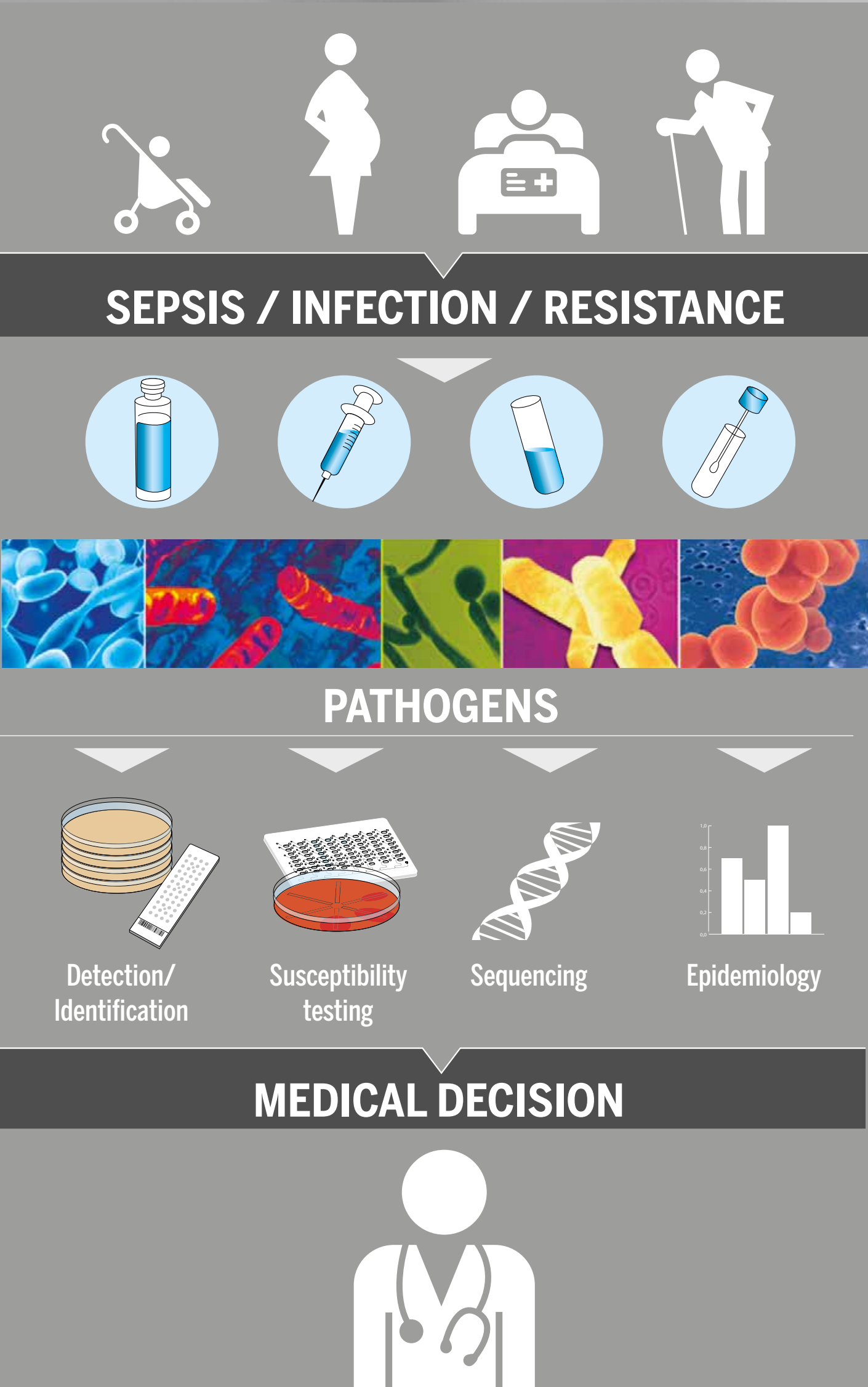

How we care more

We offer solutions for:

- Monitoring of cystic fibrosis patients
- Management of gastrointestinal infections
- Early identification of antimicrobial resistance to preserve antibiotics for future generations



CULTURE OF QUALITY

Guaranteeing consistent product quality, user and patient safety, and limited environmental impact

- Dedicated Quality Control experts in 8 production sites
- ISO 9001 (2000) and ISO 13485 (2004) certification
- Quality Control certificate for each product
 - Traceability throughout the production process
 - Inter-lot reproducibility
- Thermal Shock Statement guarantees culture media stability in real-life situations: 1 to 4 weeks at 15-25 °C
- Clinical evaluation trials provide routine use performance indicators to support all product claims

CULTURE
OF
CONFIDENCE

CULTURE
OF
QUALITY

CULTURE
OF
SERVICE

CULTURE
OF
INNOVATION



SAMPLE
TYPES

BIBLIO-
GRAPHY

CULTURE OF SERVICE

Making your daily life easier

Listening to your needs

- Centralization of customer feedback for continuous improvement
- Development of new-generation media

Helping you meet accreditation requirements

- Validated Certificates of Compatibility
- On-line Technical Library

A worldwide presence to be closer to you

- Local network of specialists
- Local training and “Education days”

CULTURE
OF
CONFIDENCE

CULTURE
OF
QUALITY

CULTURE
OF
SERVICE

CULTURE
OF
INNOVATION



SAMPLE
TYPES

BIBLIO-
GRAPHY

CULTURE OF INNOVATION

Anticipating your new challenges in ID/AST

- Pioneer in chromogenic media since 1990: hundreds of patents submitted in 166 files
- Over 10 new culture media products developed in the past 5 years
- New R&D site entirely dedicated to culture media research
- Integration of new algorithm technologies on WASPLab® empowered by bioMérieux
- Guarantee of raw material perennity and constant identification of improvement opportunities
 - Validation of a new agar source
 - Development of specific pathology media, e.g. CHROMID® S. aureus Elite for respiratory samples from cystic fibrosis patients

CULTURE
OF
CONFIDENCE

CULTURE
OF
QUALITY

CULTURE
OF
SERVICE

CULTURE
OF
INNOVATION



SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID®

The most extensive range of chromogenic media for the simultaneous culture and identification of micro-organisms

Easy to use and read

- Specific and intense coloration for clear identification
- Improved detection of mixed cultures
- Limited training needed

Quicker, reliable results for better patient treatment

- Identification in 18-24 hours
- Less need for confirmation testing

Improved workflow

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®]

VALIDATED and COMPATIBLE




WASP[®]
empowered
by bioMérieux




WASPLab[®]
empowered
by bioMérieux



VITEK[®]
SOLUTIONS

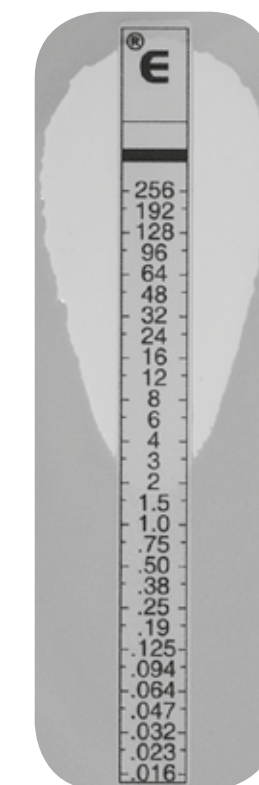

ETEST[®]

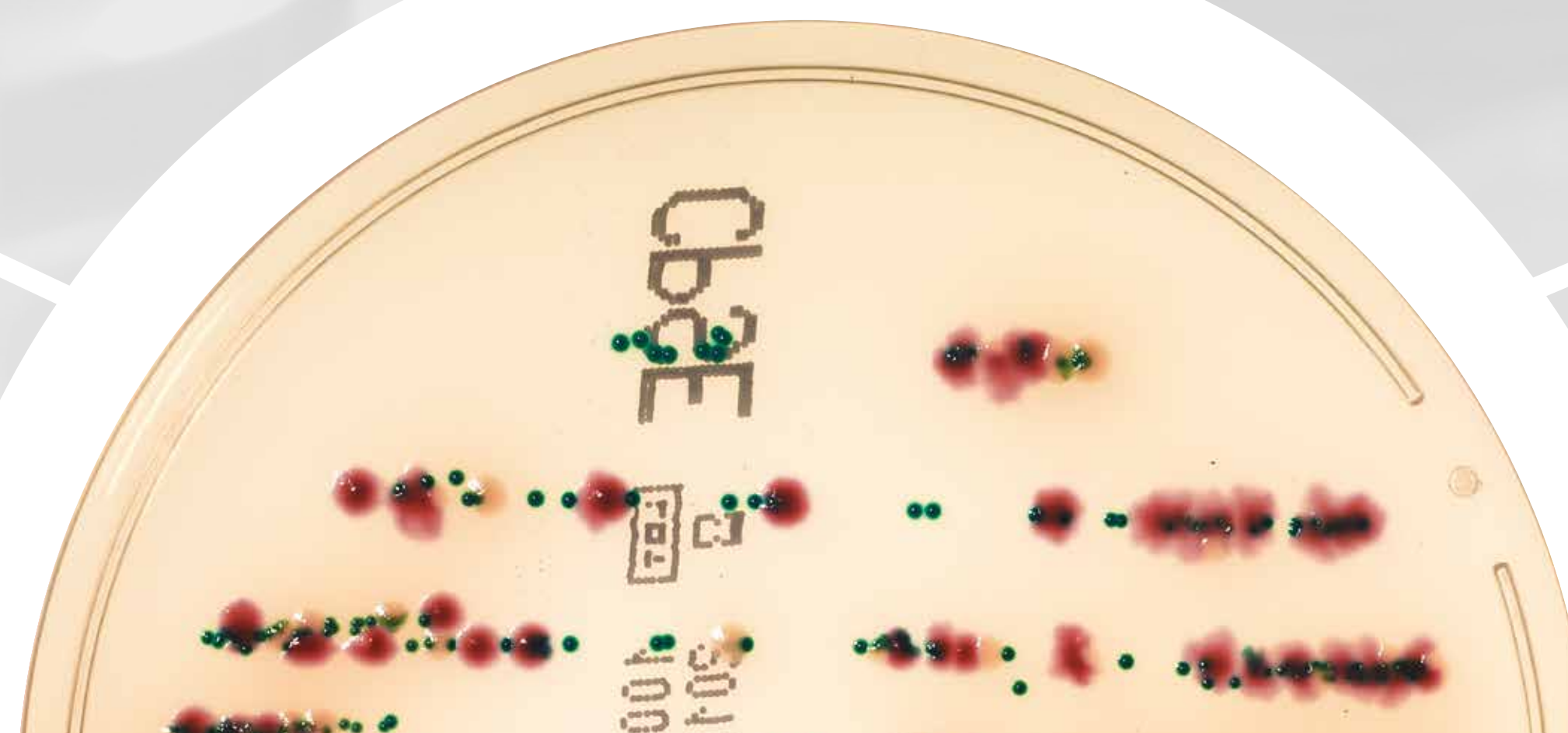

CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®] CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar

SAMPLE
TYPES


BIBLIO-
GRAPHY

CHROMID® CPS® Elite agar

Isolation, enumeration and direct or presumptive identification of urinary tract infection organisms

E.coli ATCC® 25922, *Enterococcus faecalis* ATCC® 29212, *Proteus mirabilis* ATCC® 12453

Same performance, simply choose your favorite reading

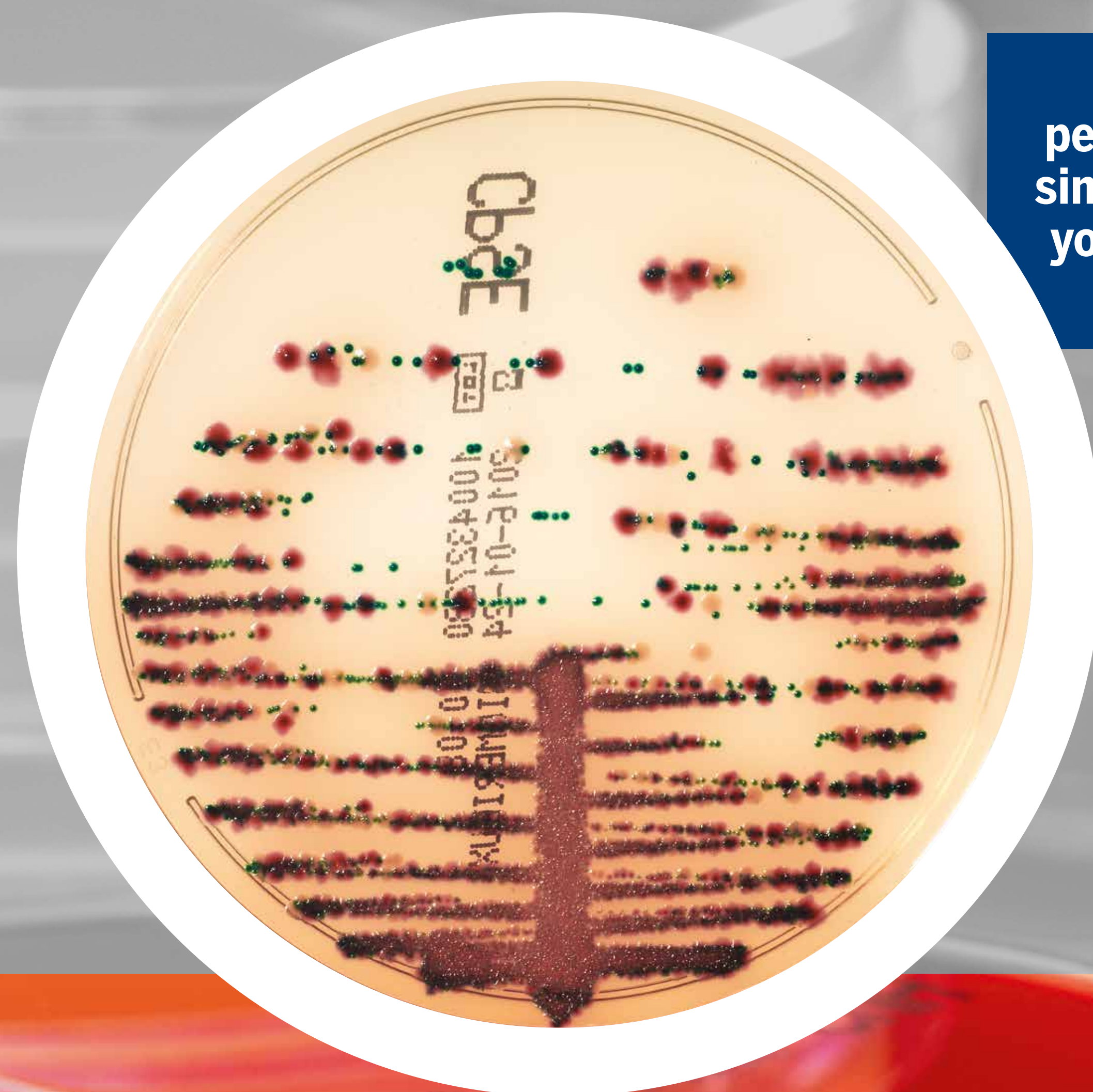

CULTURE OF ...

CHROMID®

CHROMID® CPS® Elite agar

CHROMID® CPS® Elite/ Columbia CNA + 5% sheep blood agars

CHROMID® Candida agar

CHROMID® Candida / SGC2 agars

CHROMID® S.aureus Elite agar

CHROMID® Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID® Salmonella Elite agar

CHROMID® C. difficile agar

CHROMID® P. aeruginosa agar

CHROMID® Vibrio agar



SAMPLE TYPES

BIBLIOGRAPHY

CHROMID[®] CPS[®] Elite agar



- Validated for reading as of 16 hours
- 2-week storage at 15-25°C
- Enhanced colony coloration
- Enumeration protocol validated with 1 µL or 10 µL

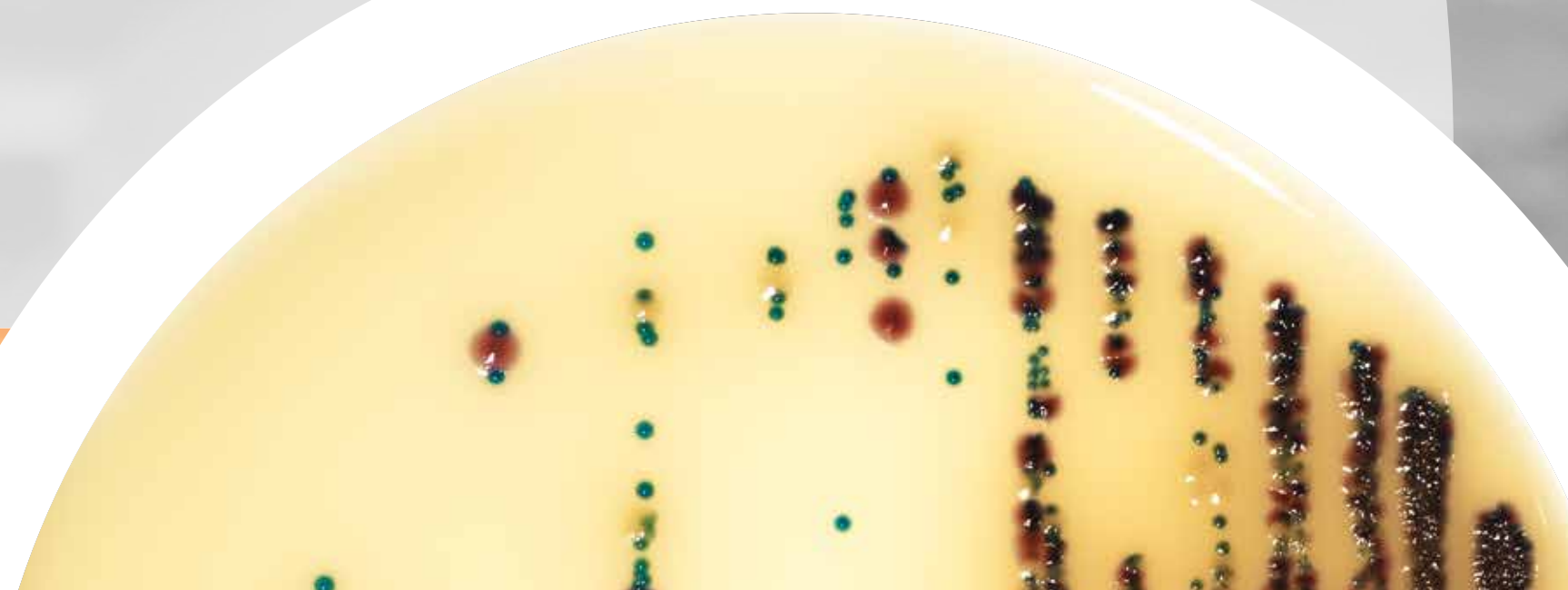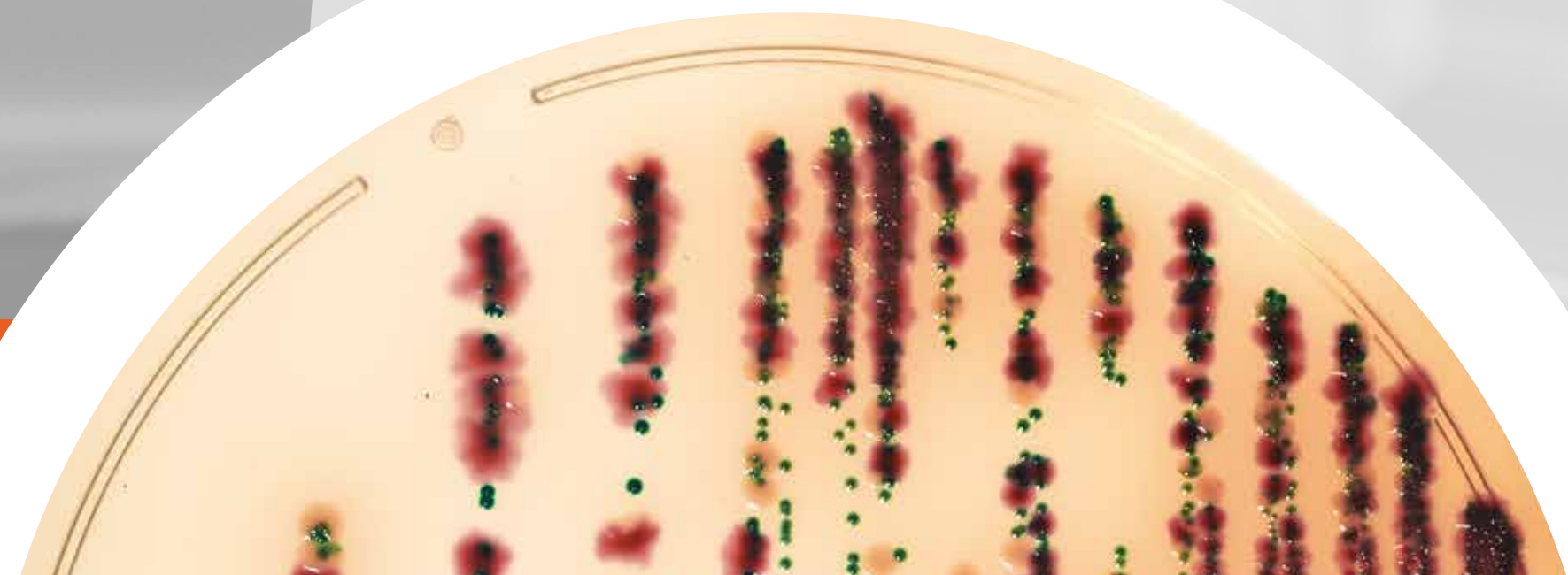

CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY


CHROMID® CPS® Elite/Columbia CNA + 5% sheep blood agars

Isolation, enumeration and direct or presumptive identification of urinary tract infection organisms

Selective isolation of fastidious bacteria

Determination of hemolysis

E.coli ATCC® 25922

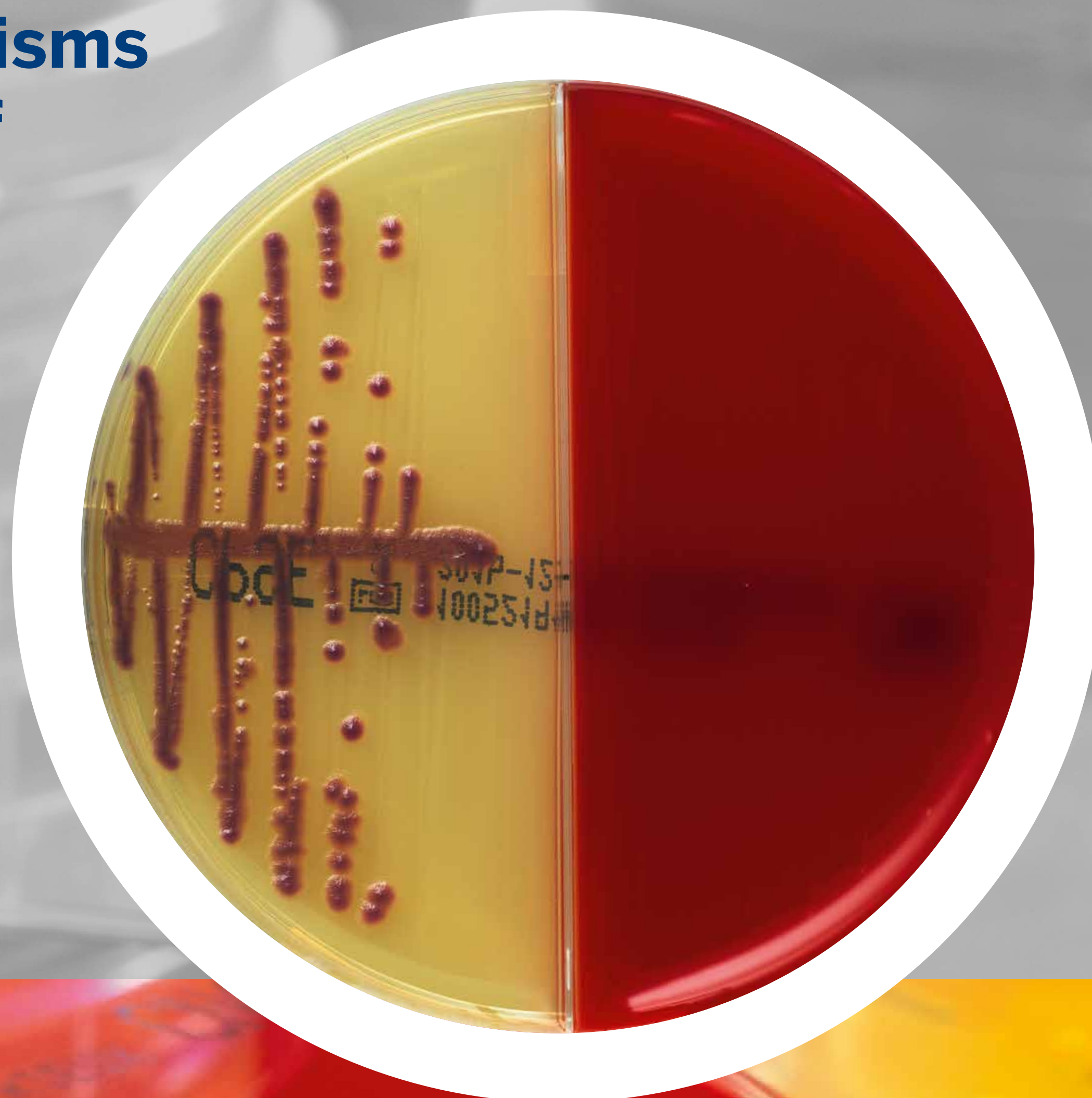

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] CPS[®] Elite/Columbia CNA + 5% sheep blood agars



- Only 1 handling step to test 2 media
- In pregnant women:
Simultaneous detection of all pathogens involved in urinary infections + hemolytic Group B Streptococci
- Reduce costs through improved stock and waste management



CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] Candida agar

Selective isolation of yeasts and direct identification of *Candida albicans*

Candida albicans
Candida tropicalis

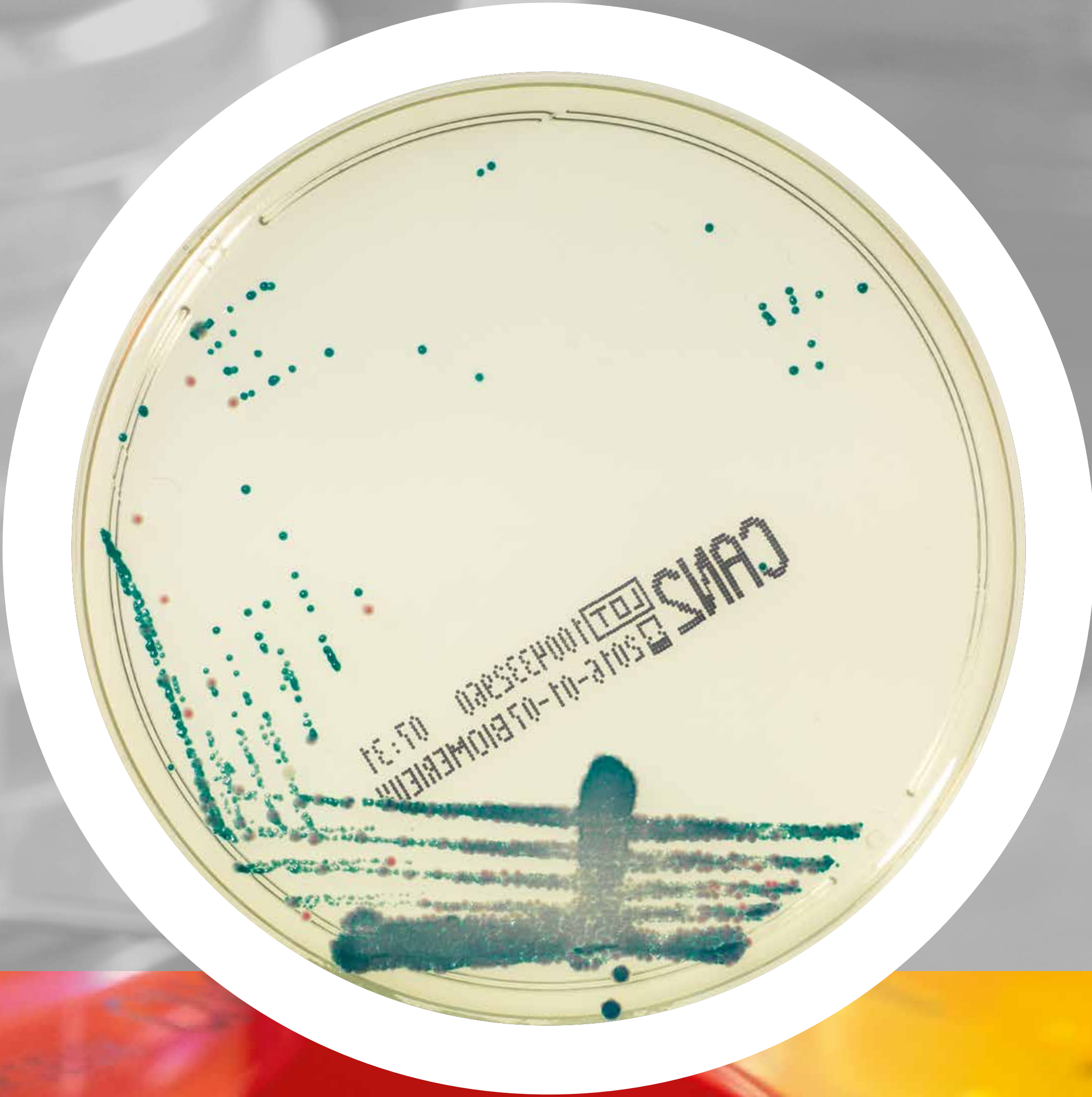

CULTURE OF ...

CHROMID[®]

CHROMID[®] CPS[®] Elite/Elite agar

CHROMID[®] CPS[®] Elite/Columbia CNA + 5% sheep blood agars

CHROMID[®] Candida /SGC2 agars

CHROMID[®] Strepto B agar

CHROMID[®] S.aureus Elite agar

CHROMID[®] Candida agar

CASA[™] agar

GRANADA[™] agar

CHROMID[®] Salmonella Elite agar

CHROMID[®] C. difficile agar

CHROMID[®] P. aeruginosa agar

CHROMID[®] Vibrio agar


SAMPLE TYPES

BIBLIOGRAPHY

CHROMID[®] Candida agar



- Direct identification of *C. albicans* in only 24 hours
- Growth of *C. auris* and *C. krusei*
- Characteristic appearance of filamentous fungi
- Easy detection of mixed cultures

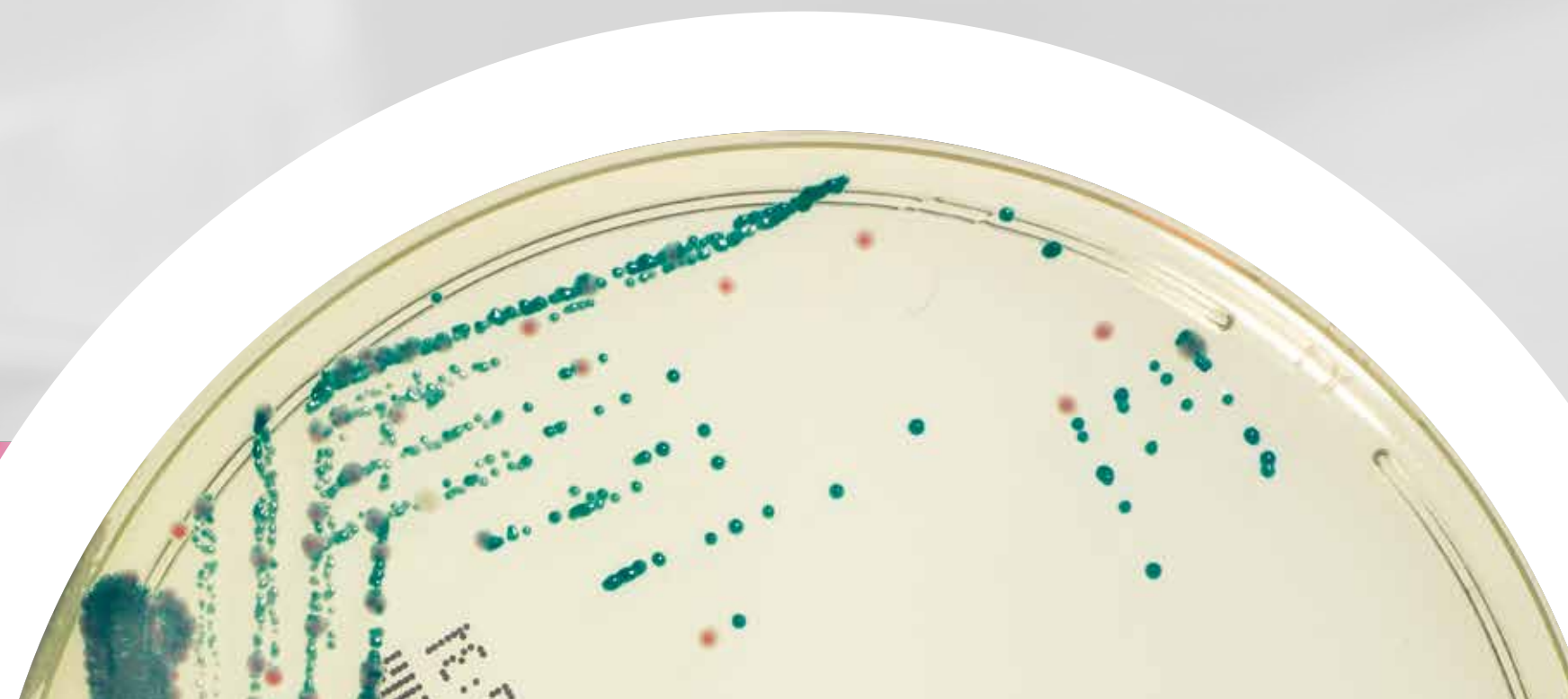

CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] Candida / SGC2^{*} agars

Selective isolation of yeasts, molds and direct identification of *Candida albicans*



* Sabouraud Gentamicin Chloramphenicol 2

CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® Candida / SGC2* agars



- Only 1 handling step to test 2 media
- Growth of *C. auris* and *C. krusei*
- Characteristic appearance of filamentous fungi
- Reduce costs through improved stock and waste management

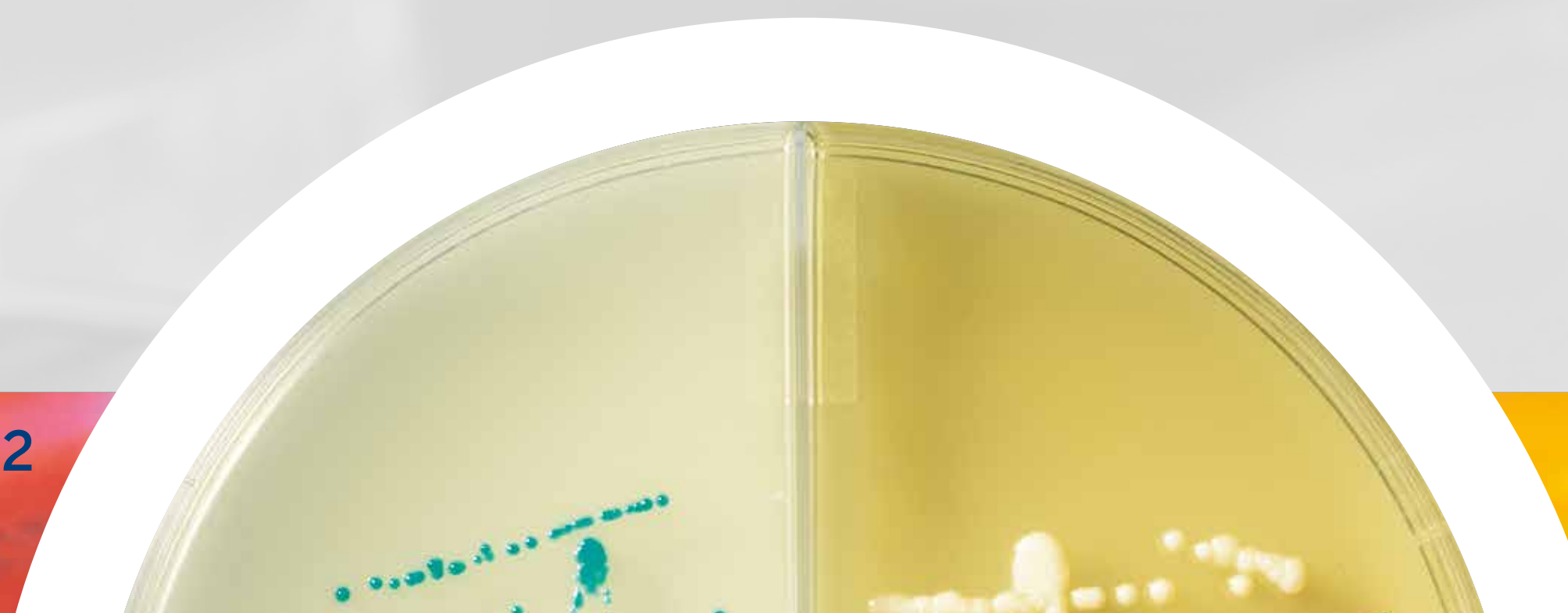

* Sabouraud Gentamicin Chloramphenicol 2



CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar


CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] S. aureus Elite agar

Selective isolation and direct identification
of *Staphylococcus aureus*

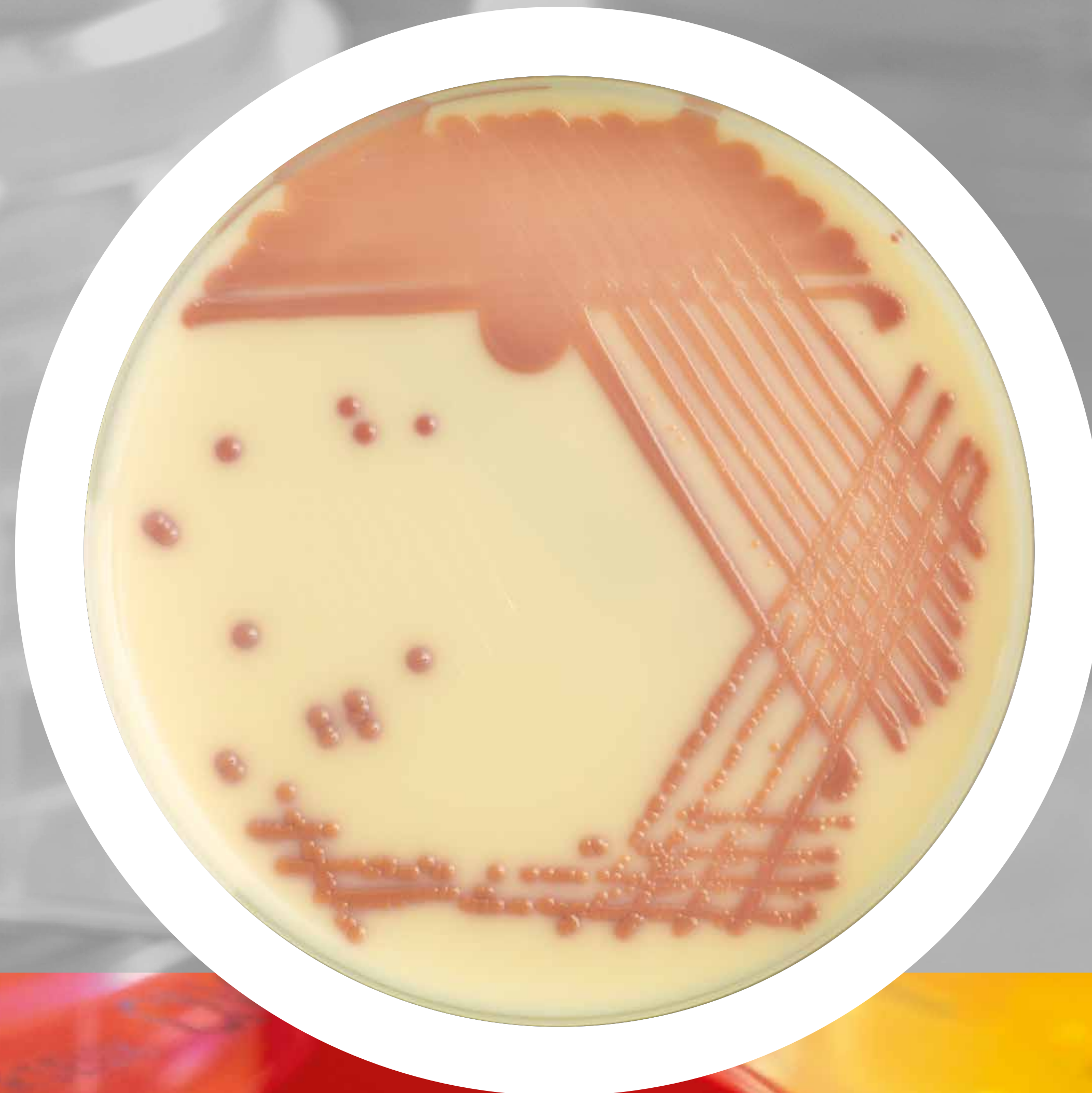

CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] *S. aureus* Elite agar



- Medium enriched with thymidine for detection of *S. aureus*, including Small Variant Colonies
- Validated for monitoring cystic fibrosis respiratory samples
- Validated for screening and diagnosis on many, medically valuable samples

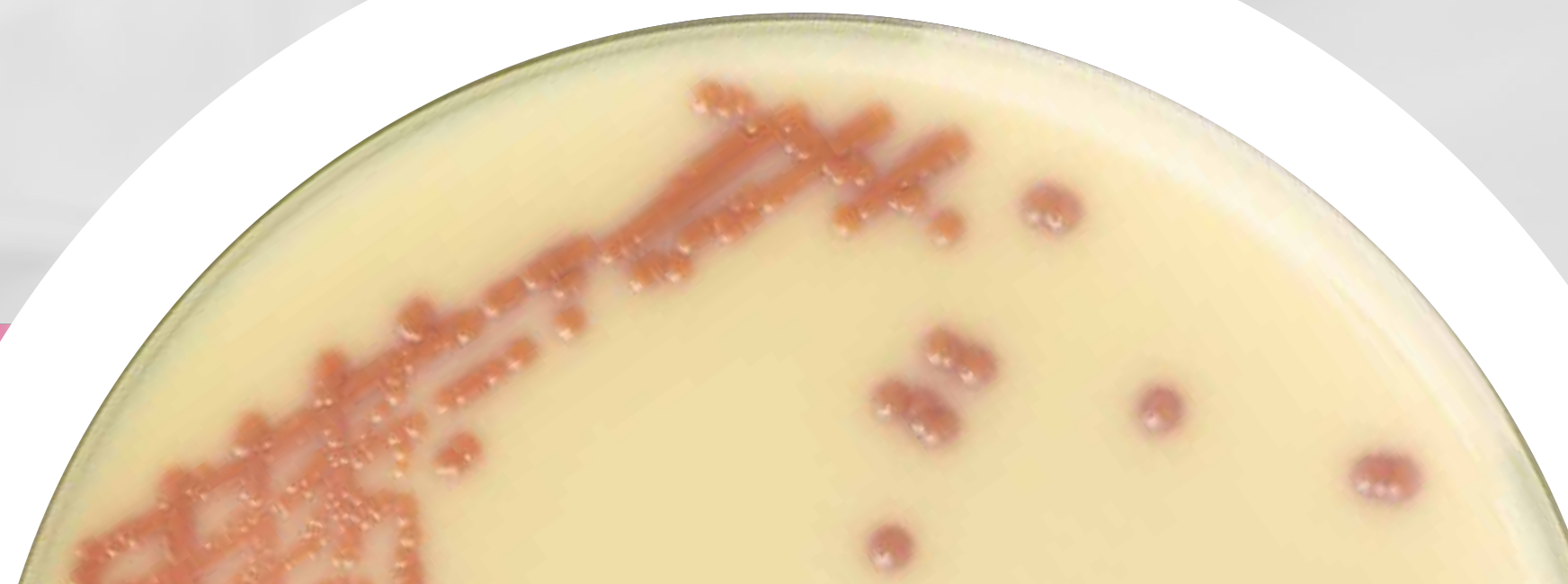

CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S. aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® Strepto B agar

Screening of Group B Streptococci (*Streptococcus agalactiae*)
in pregnant women



CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® Strepto B agar



- In pregnant women: Highly sensitive detection of Group B Streptococci, including non-hemolytic strains
- Optimized balance between selectivity and fertility → less confirmatory tests
- Results you can trust: numerous scientific proof^{1,2,3}
- Incubation in normal atmosphere

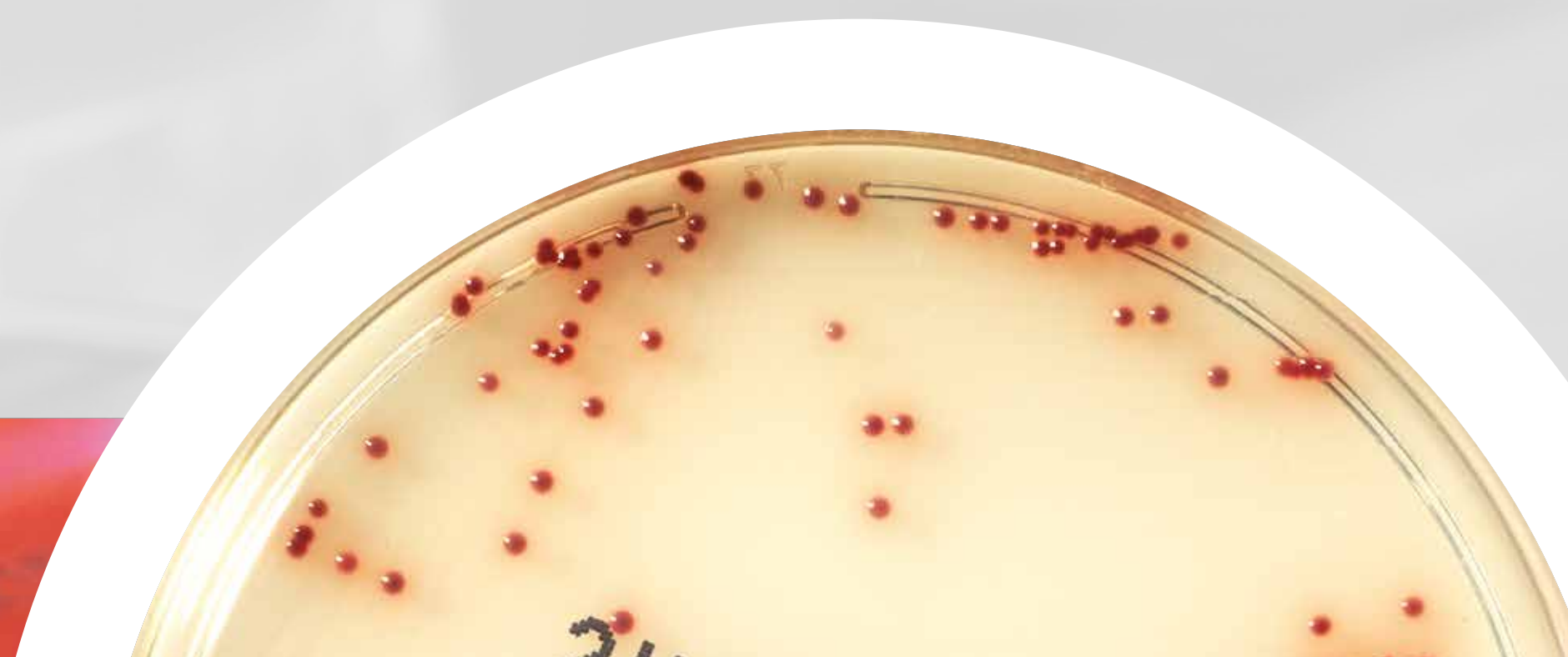

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar


CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

GRANADA™ agar

Screening and identification of Group B Streptococci
(*Streptococcus agalactiae*) in pregnant women

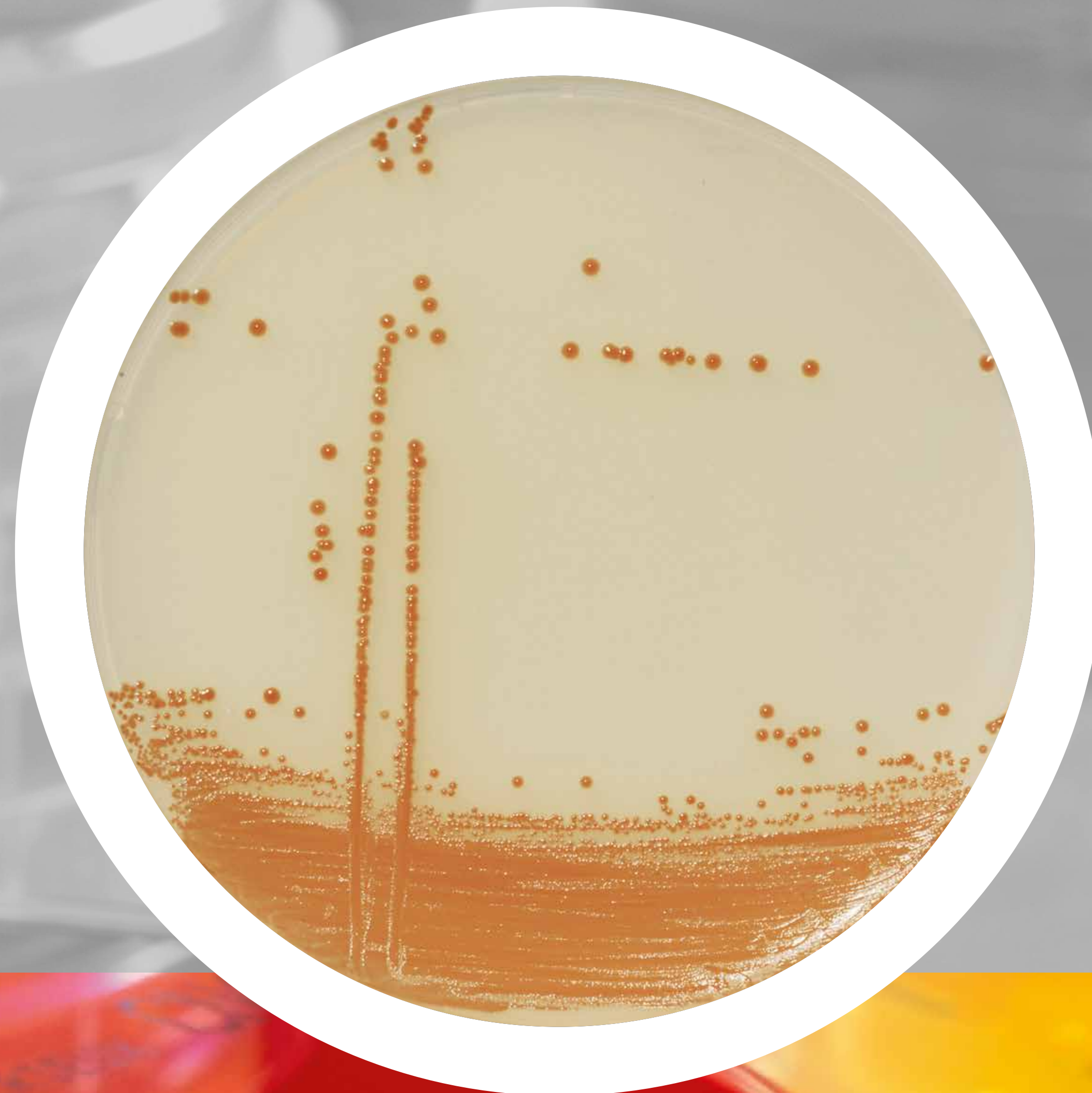

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

GRANADA™ agar



- In pregnant women:
Direct identification of hemolytic Group B Streptococci in 18 hours
- Rapid solution to optimize your workflow
- Results you can trust: numerous scientific proof^{2,4,5}



CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CASA™ agar

Selective medium for the detection and enumeration of *Campylobacter*



CULTURE OF ...

CHROMID®

CHROMID® CPS® Elite/ Columbia CNA + 5% sheep blood agars

CHROMID® SGC2 agars

CHROMID® Strepto B agar

CASA™ agar

CHROMID® C. difficile agar

CHROMID® Vibrio agar

CHROMID® CPS® Elite agar

CHROMID® Candida agar

CHROMID® S.aureus Elite agar

GRANADA™ agar

CHROMID® Salmonella Elite agar

CHROMID® P. aeruginosa agar


SAMPLE TYPES

BIBLIO-GRAPHY

CASA™ agar



- The only chromogenic medium available for *Campylobacter*
- Detects at least 10% more *Campylobacter* than conventional media with earlier reading
- High selectivity → Less unnecessary confirmatory tests
- Validation of performance at 37°C and 42°C

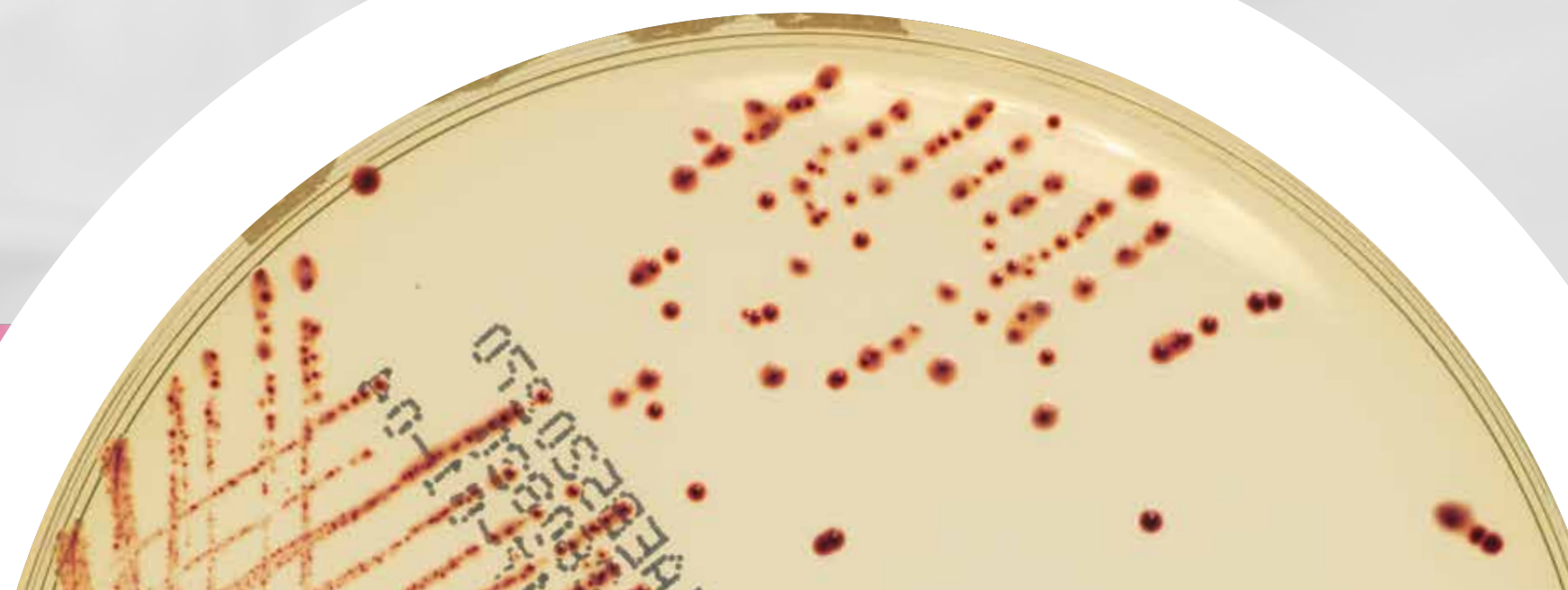

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] Salmonella Elite agar

Selective isolation and
direct identification of *Salmonella* species



CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] Salmonella Elite agar



- Growth of *Salmonella* including *S. arizonae* & *S. Typhi*
- High selectivity → Less unnecessary confirmatory tests
- Better contrast with opacity → Easy reading



CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar


CHROMID[®]
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® C. difficile agar

Detection and identification
of *Clostridium difficile*

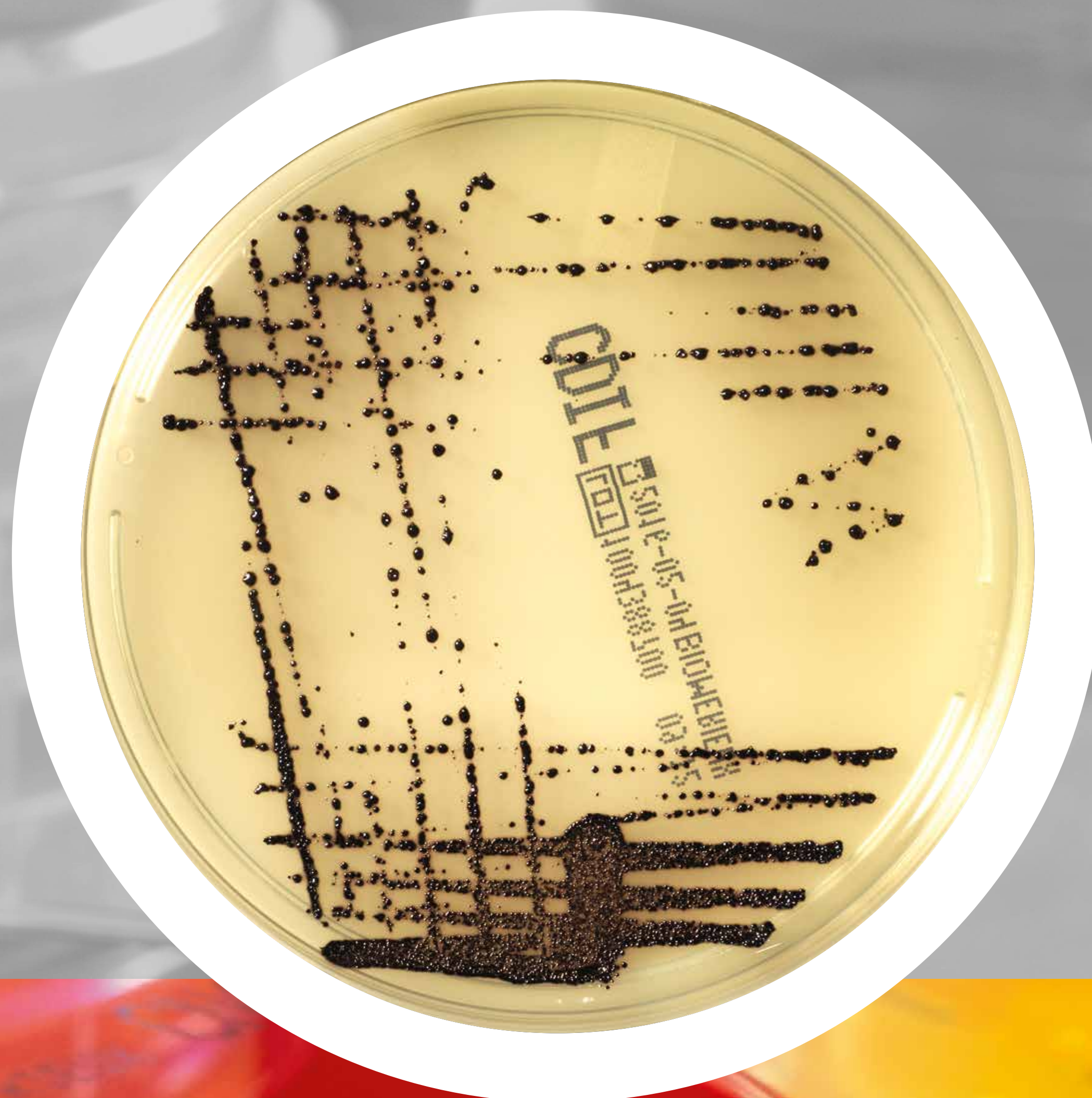

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] C. difficile agar



- Higher sensitivity: includes taurocholate for spore germination
- Result within 24 hours for earlier detection of *C. difficile*
- Results you can trust: numerous scientific proof^{6,7,8}



CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID[®] P. aeruginosa agar

Direct identification of *Pseudomonas aeruginosa*



CULTURE
OF
...

CHROMID[®]

CHROMID[®] CPS[®]
Elite agar

CHROMID[®]
CPS[®] Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID[®]
Candida agar

CHROMID[®]
Candida /
SGC2 agars

CHROMID[®]
S.aureus Elite
agar

CHROMID[®]
Strepto B agar

GRANADA[™] agar

CASA[™] agar

CHROMID[®]
Salmonella
Elite agar

CHROMID[®]
C. difficile agar

CHROMID[®]
P. aeruginosa
agar

CHROMID[®]
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® *P. aeruginosa* agar



- Validated for monitoring cystic fibrosis respiratory samples
- Detection of all morphotypes of *P. aeruginosa*
- Direct identification in 18-24 hours

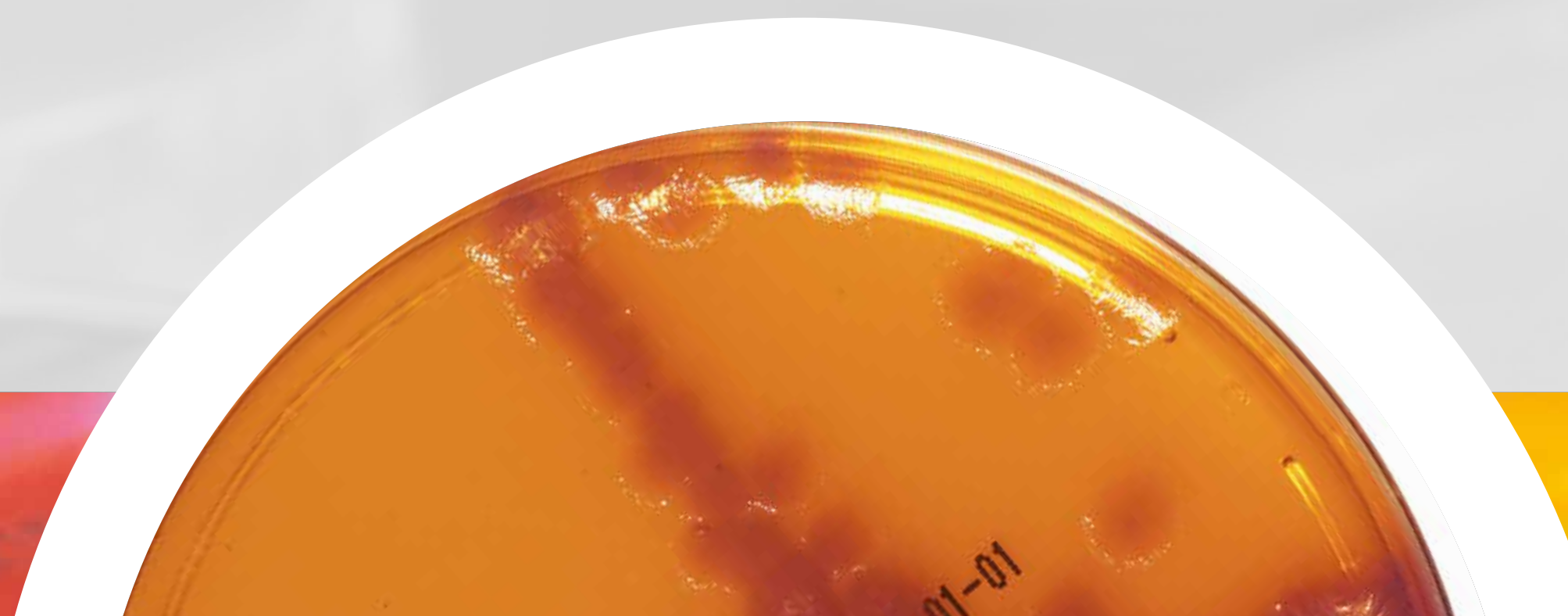

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® Vibrio agar

Selective isolation of *Vibrio* and presumptive identification of *V. cholerae* and *V. parahaemolyticus*



CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

CHROMID® Vibrio agar



- Colour differentiation between *V. cholerae* and *V. parahaemolyticus*
- Validated on food and clinical samples
- Detection in 24 hours

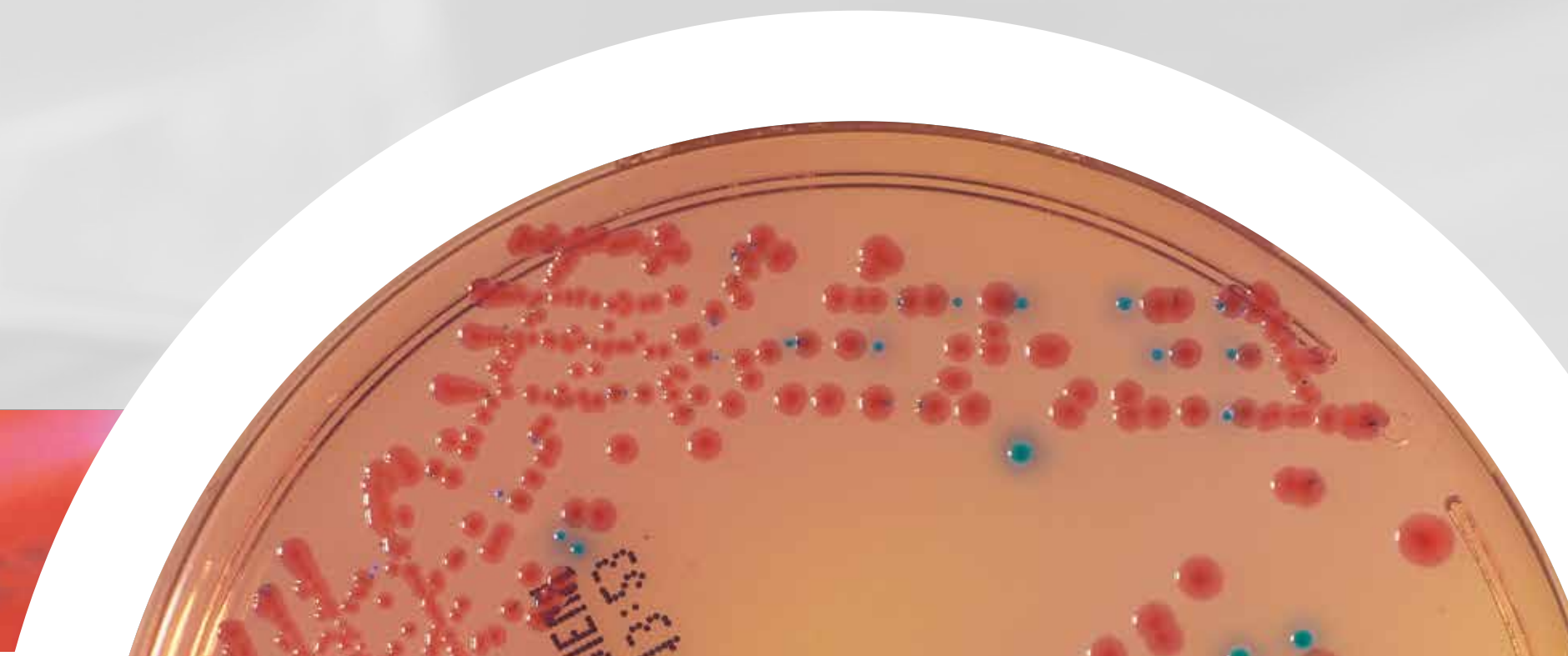

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar


CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

Managing different sample types

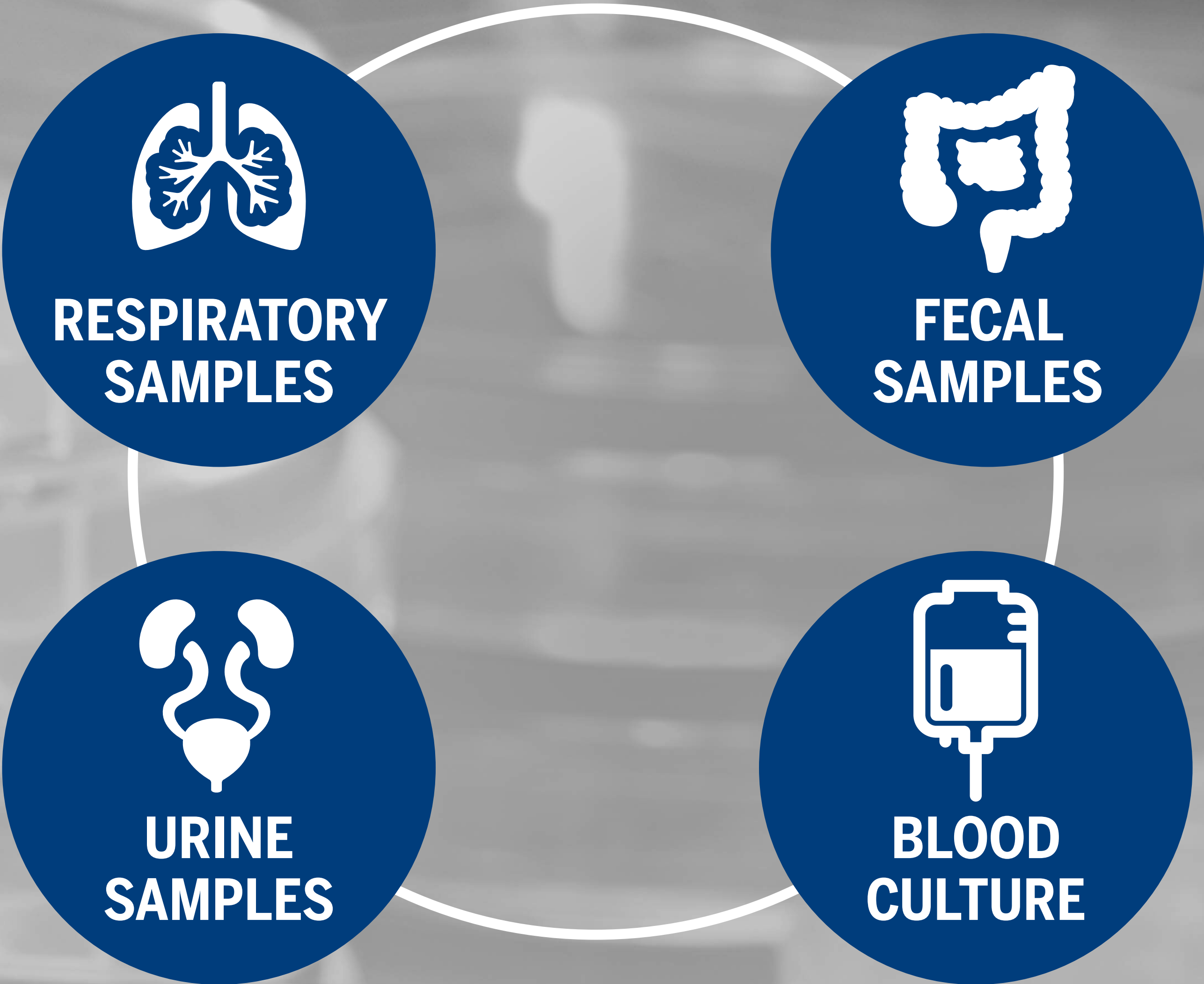

CULTURE OF ...

CHROMID®

CHROMID® CPS® Elite/ Columbia CNA + 5% sheep blood agars

CHROMID® CPS® Elite agar

CHROMID® Candida / SGC2 agars

CHROMID® Candida agar

CHROMID® S.aureus Elite agar

CHROMID® Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID® Salmonella Elite agar


CHROMID® C. difficile agar

CHROMID® P. aeruginosa agar

CHROMID® Vibrio agar

SAMPLE TYPES

BIBLIO-GRAPHY

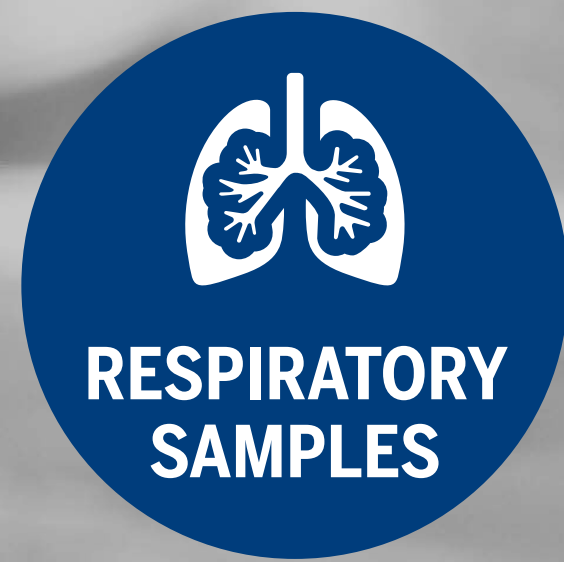


**Full range of
conventional media**



**CHROMID®
P. aeruginosa agar**



**CHROMID®
S. aureus Elite agar**

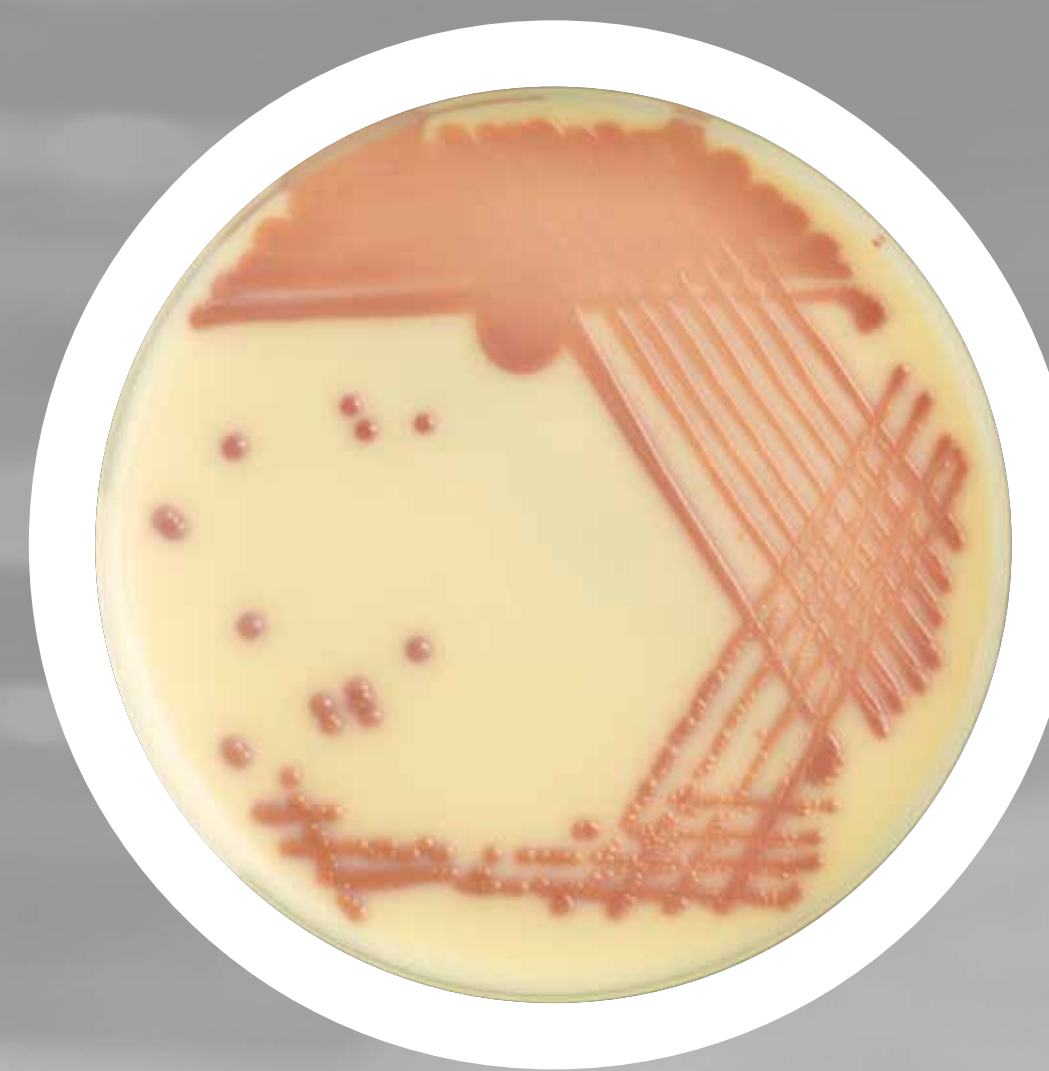

**CHROMID®
Candida agar**



CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S. aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar


CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

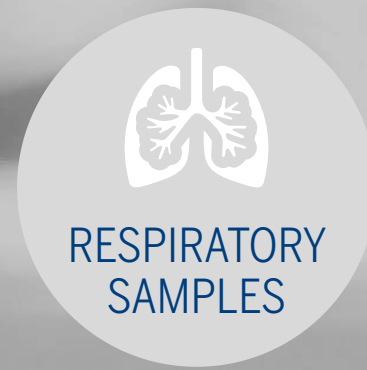

RESPIRATORY
SAMPLES



FECAL
SAMPLES



URINE
SAMPLES



BLOOD
CULTURE

**Full range of
conventional media**



**CHROMID®
Salmonella Elite agar**



CASA™ agar




Complementary testing for specific cases:

**CHROMID®
Vibrio agar**




**CHROMID®
S. aureus Elite agar**

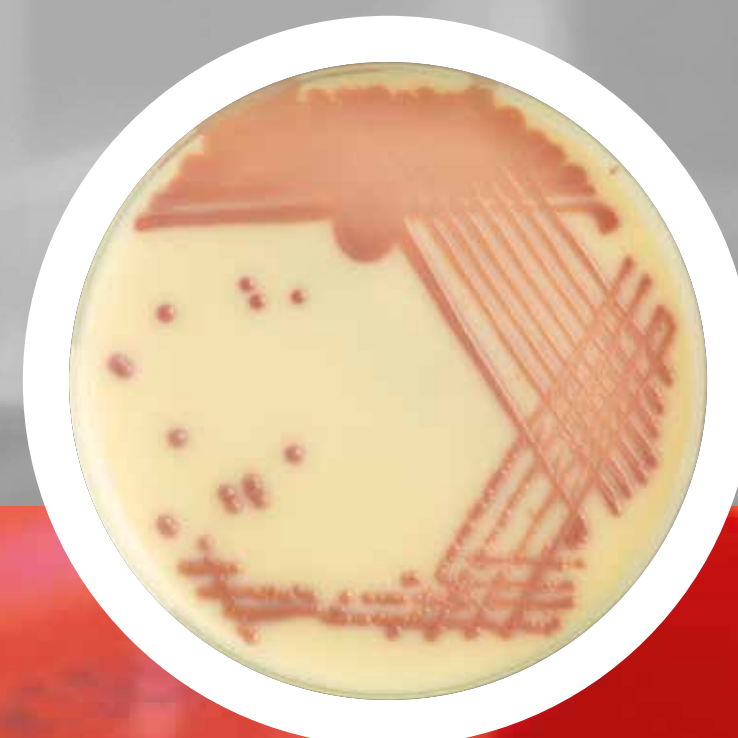

**CHROMID®
Candida agar**



**CHROMID®
C. difficile agar**

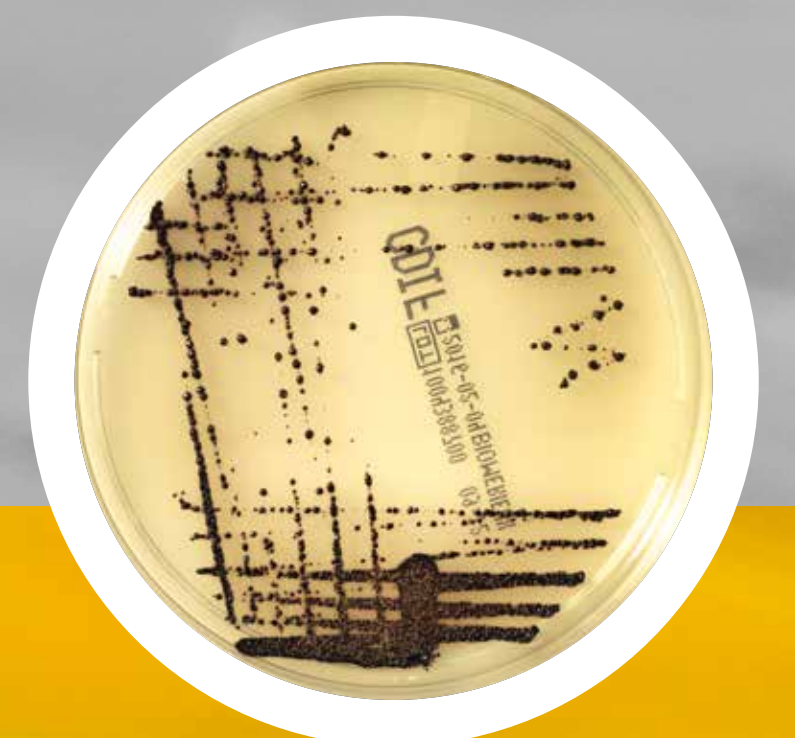

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar


CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar


SAMPLE
TYPES

BIBLIO-
GRAPHY

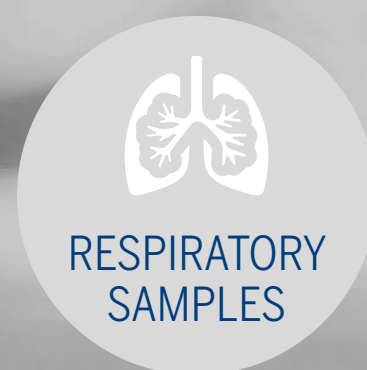

CHROMID® CPS® Elite agar

**Transparent
or opaque:**
choose your
favourite reading
for the same
performance

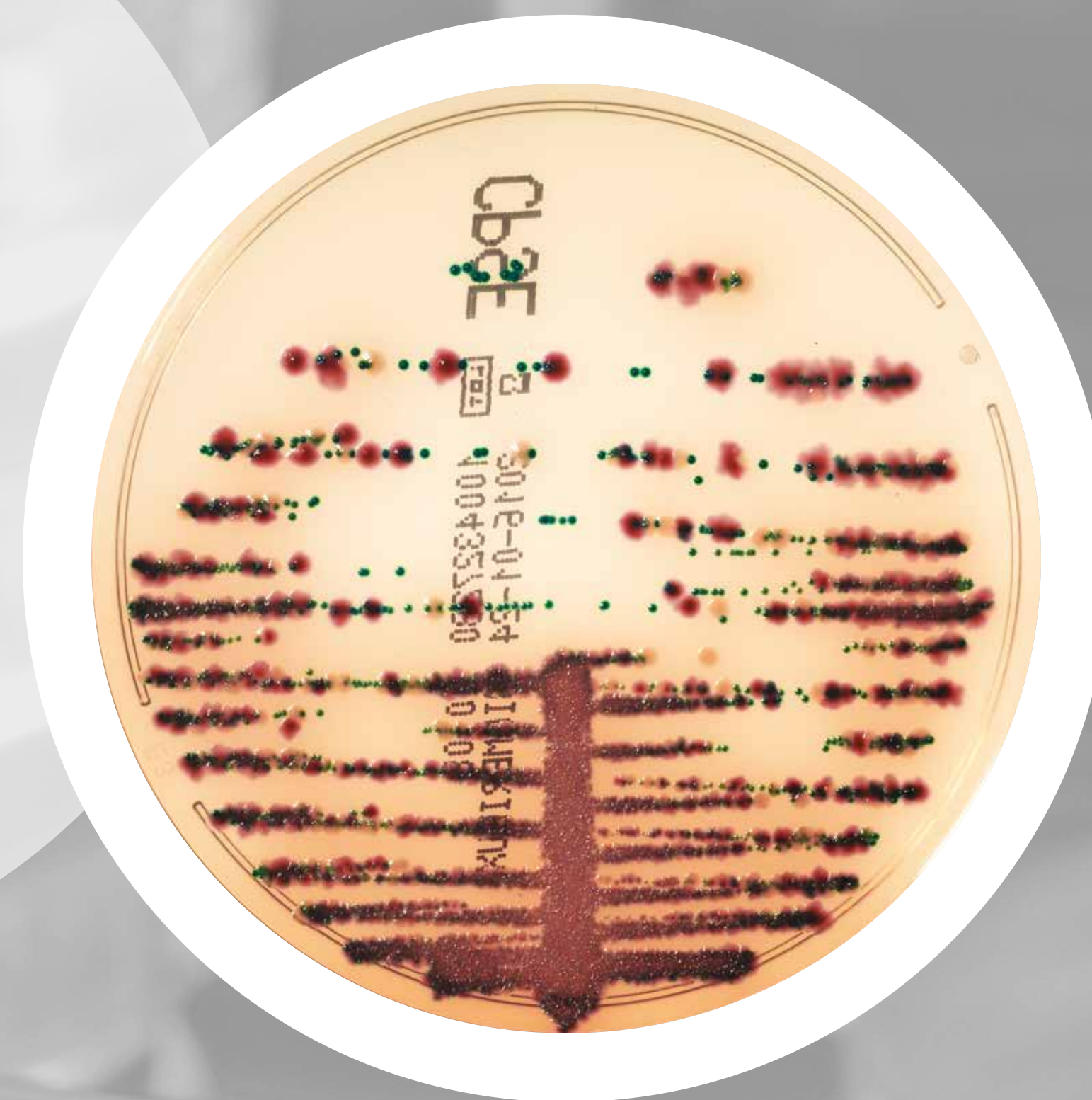

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar


CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

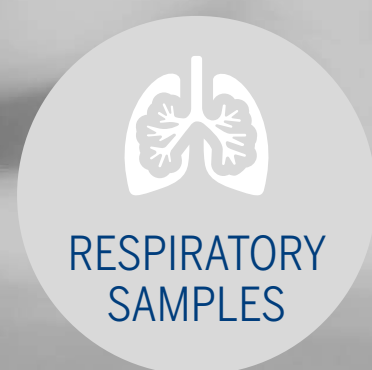


Full range of conventional media



CHROMID® MRSA agar



CHROMID® S. aureus Elite agar

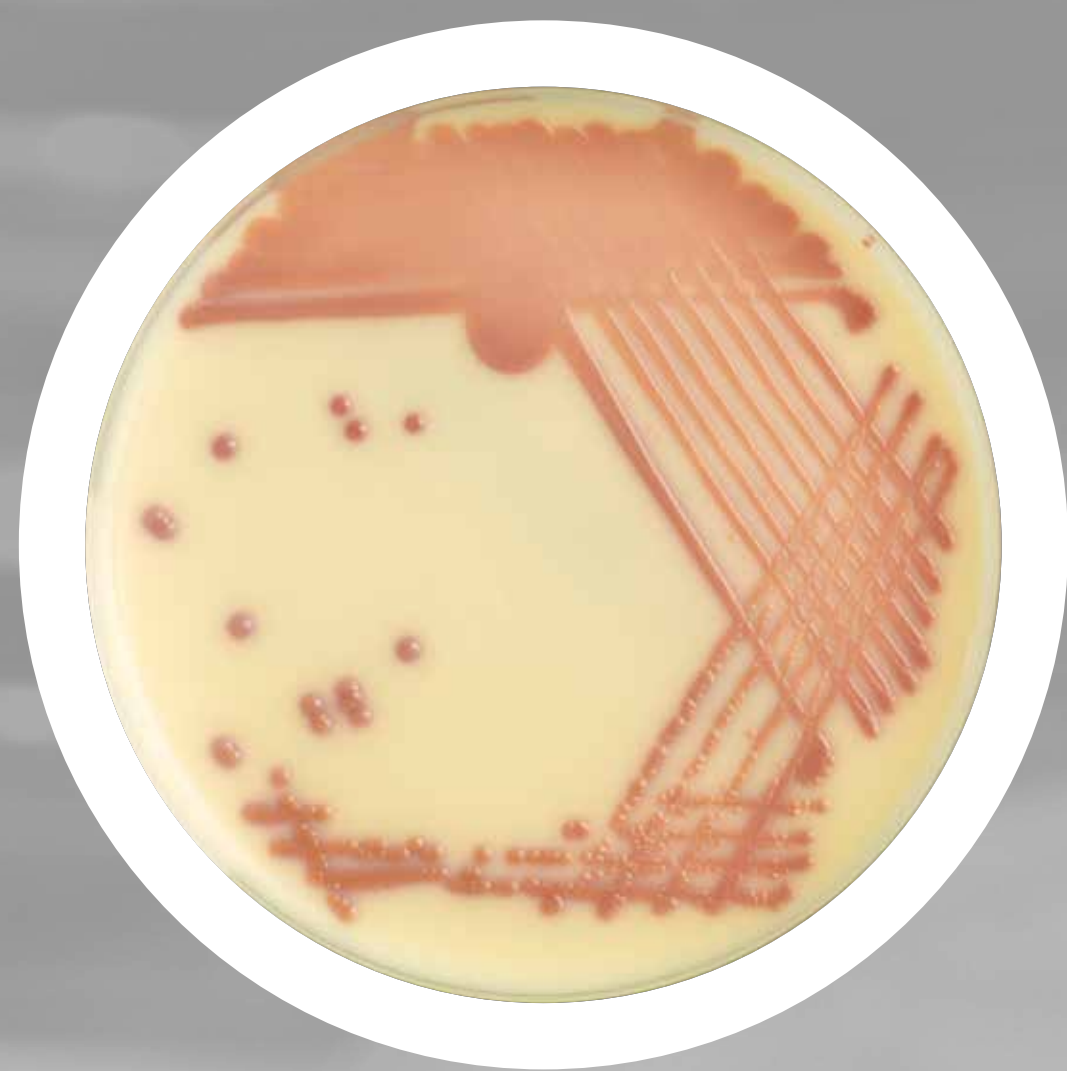

CHROMID® Candida agar



CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY

BIBLIOGRAPHY

07-18 / Demonstration and/or pictures not legally binding. Modifications by bioMérieux can be made without prior notice / BIOMÉRIEUX, the BIOMÉRIEUX logo, CHROMID, ETEST and VITEK are used, pending and/or registered trademarks belonging to bioMérieux or one of its subsidiaries or one of its companies / WASP and WASPLab are trademarks belonging to Copan Italia S.p.A. / Any other name or trademark is the property of its respective owner / bioMérieux S.A. RCS Lyon 673 620 399 / Photos: N. Bouchut, C. Ganet, Getty Images, bioMérieux / Printed in France / Théra RCS Lyon B 398 160 242

1. Morita T, *et al.* Evaluation of CHROMID® Strepto B as a screening media for *Streptococcus agalactiae*. BMC Infect Dis. 2014 Jan 29;14:46. doi: 10.1186/1471-2334-14-46.
2. Verhoeven PO, *et al.* Evaluation of the New Brilliance GBS Chromogenic Medium for Screening of *Streptococcus agalactiae* Vaginal Colonization in Pregnant Women. J Clin Microbiol. 2014 Mar;52(3):991-3. doi: 10.1128 / JCM.02926-13. Epub 2014 Jan 8.
3. Binghuai L, *et al.* Use of MALDI-TOF mass spectrometry for rapid identification of group B Streptococcus on chromID Strepto B agar. Int J Infect Dis. 2014 Oct;27:44-8. doi: 10.1016/j.ijid.2014.06.023. Epub 2014 Aug 23.
4. Romanik M, *et al.* Identification of Pregnant women at risk of Streptococcus group B colonisation. Lett 2011; 32 (3):308-312.
5. Fulchiron C, *et al.* Evaluation of the performance of GRANADA medium (bioMérieux) under CO2 atmosphere for automated incubation. Poster presented at ECCMID 2016.
6. Smits WK, *et al.* *Clostridium Difficile* infection. Nat Rev Dis Primers. 2016 Apr 7;2:16020. doi: 10.1038/nrdp.2016.20.
7. Chen HKJ, *et al.* The importance of matrix-assisted laser desorption ionization - time of flight mass spectrometry for correct identification of *Clostridium difficile* isolated from chromID C. difficile chromogenic agar. J Microbiol Immunol Infect. 2016 Jan 11. pii: S1684-1182(16)00003-7. doi: 10.1016/j.jmii.2015.12.002.
8. Han SB, *et al.* Performance of CHROMID Clostridium difficile agar compared with BBL C. difficile selective agar for detection of C. difficile in stool specimens. Ann Lab Med. 2014 Sep;34(5):376-9. doi: 10.3343/alm.2014.34.5.376. Epub 2014 Aug 21.

CULTURE
OF
...

CHROMID®

CHROMID® CPS®
Elite agar

CHROMID®
CPS® Elite/
Columbia CNA
+ 5% sheep
blood agars

CHROMID®
Candida agar

CHROMID®
Candida /
SGC2 agars

CHROMID®
S.aureus Elite
agar

CHROMID®
Strepto B agar

GRANADA™ agar

CASA™ agar

CHROMID®
Salmonella
Elite agar

CHROMID®
C. difficile agar

CHROMID®
P. aeruginosa
agar

CHROMID®
Vibrio agar

SAMPLE
TYPES

BIBLIO-
GRAPHY